



PRID[®] DELTA FLYING HIGH IN FERTILITY

APPLICATION OF ANY TYPE OF PRODUCT HAS A DIRECT EFFECT ON ITS EFFICACY. MAKE SURE YOU ADDRESS ALL OF THE POINTS IN THIS CHECKLIST EVERY TIME YOU USE PRID[®] DELTA TO OBTAIN THE BEST RESULTS!

- ▶ Soak **PRID[®] DELTA** applicator in non-spermicidal disinfectant for at least 5 minutes prior to **PRID[®] DELTA** insertion
- ▶ Use 2 or more applicators to enable 5 minute contact time with disinfectant between uses
- ▶ Make sure you have checked there is no pre-existing metritis prior to **PRID[®] DELTA** insertion
- ▶ Disinfect vulva with a diluted disinfectant i.e. chlorhexidine or iodine prior to **PRID[®] DELTA** insertion
- ▶ Insert the **PRID[®] DELTA** device into the applicator just before insertion
- ▶ Use new clean gloves to handle applicator and **PRID[®] DELTA** each time
- ▶ Consider reducing the length of the **PRID[®] DELTA** tail particularly if heifers are housed collectively. Cut the **PRID[®] DELTA** tail 5 - 10cm outside the vulva ensuring to leave sufficient length for an easy removal of the device.

For support and further information on ruminant reproduction management speak to your Ceva Livestock Territory Manager or our technical team on 01494 781510

PRID[®] DELTA 1.55g vaginal delivery system for cattle. **COMPOSITION:** Each device contains 1.55g progesterone. Please refer to the product packaging and leaflets for information about side effects, precautions, warnings and contra-indications. Legal category: **POM-V**

Prescription decisions are for the person issuing the prescription alone. Please use medicines responsibly (www.noah.co.uk/responsible)


Ceva Animal Health Ltd, Explorer House, Mercury Park, Wycombe Lane, Wooburn Green, Bucks. HP10 0HH Tel: 01628 334056
www.ceva.com



PRID[®] DELTA INSERTION GUIDE



PRID[®] DELTA

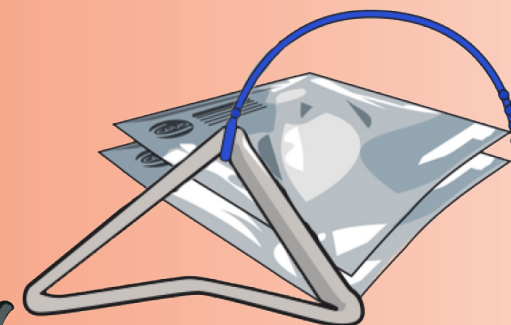

APPLICATOR

ONCE THE COW IS READY START WITH THE PRID DELTA PREPARATION

Intravaginal devices can lose elasticity if they are left in the applicator for too long.

